

Avian Influenza Virus Ag Test

Cat No.: W81033

1. INTENDED USE

Quicking Avian Influenza Virus Antigen Rapid Test is a sandwich lateral flow immunochromatographic assay for the qualitative detection of avian influenza virus (AIV Ag) in avian secretions.

Assay Time: 5 - 10 min

Sample: secretions or serum


2. PRINCIPLE OF THE ASSAY

Quicking Avian Influenza Virus Ag Rapid Test is based on sandwich lateral flow immunochromatographic assay. The test device has a testing window. The testing window has an invisible T (test) zone and C (control) zone. When sample is applied into the sample hole on the device, the liquid will laterally flow on the surface of the test strip. If there is enough AIV antigen in the sample, a visible T band will appear. The C band should always appear after a sample is applied, indicating a valid result. By this means, the device can accurately indicate the presence of Avian Influenza Virus antigen in the sample.

3. KIT COMPONENT

- 10×foil pouches, each containing a cassette, pipette and desiccant
- 10×assay buffer tubes (0.8 mL each)
- 10×swab sticks
- Product Manual


4. TEST PROCEDURE

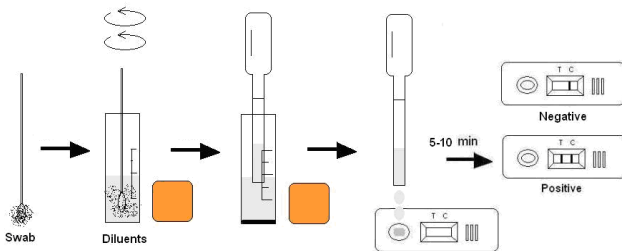

- Insert the swab stick into bird's cloacae or dip the swab stick in bird's eyes or nose to collect the secretions. Please make the swab wet sufficiently. Or take serum as sample.
- Insert the wet swab into the provided assay buffer tube. Agitate it to assure good sample extraction. Or drip 2-3 drops serum into assay buffer, mix well as sample extraction.
- Take out the cassette from the foil pouch and place it horizontally.
- Gradually drip 3 drops of sample extraction into the sample hole. If the migration has not appeared after 30 seconds, add one more drop of the sample extraction in to sample hole.

- Interpret the result in 5 - 10 minutes. Result after 10 minutes is only considered as reference.

5. INTERPRETATION OF RESULTS

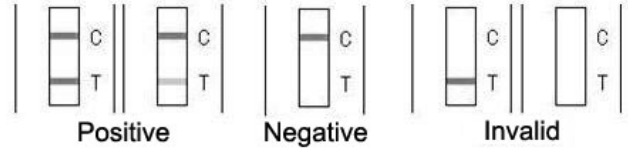

Positive: The presence of both C band and T band, no matter T band is clear or vague.

Negative: Only clear C band appears.

Invalid: No colored band appears in C zone, no matter whether T band appears.

6. STORAGE

The kit can be stored at room temperature (2-30°C). The test kit is stable through the expiration date (18 months) marked on the foil pouch. **DO NOT FREEZE.** Do not store the test kit in direct sunlight.

7. PRECAUTIONS

- For best results, please strictly adhere to these instructions.
- All reagents must be at room temperature before running the assay.
- Do not remove test cassette from its pouch until immediately before use.
- Please do not use tap water, purified water or distilled water as negative control.
- Do not reuse the test kit.
- Do not use the test beyond its expiration date marked on the foil pouch.
- The components in this kit have been quality control tested as standard batch unit. Do not mix components from different lot numbers.

8. LIMITATION

Quicking Avian Influenza Virus Antigen Rapid Test is for in vitro veterinary diagnosis use only. All results should be considered with other clinical information available with veterinarian. For an accurate result, we suggest you to choose other method such as PCR for final determination in practice.

Quicking Biotech Co., Ltd.

No. 1998, South Yanggao Road, Shanghai, China.

Tel: 86-21-5094 7188 Fax: 86-21-5094 7186

Website: www.quicking.cn Email: int_sales@quicking.cn